XO FLOW

CONTENT

EXTRAORDINARY DENTISTRY	6
MODULAR AND FUTURE PROOF	10
DENTISTRY MADE SIMPLE AND EASY	22
NEW TRENDS	72
ADVANCED MULTI-USER DENTISTRY	74
XO CARE	90

EXTRAORDINARY DENTISTRY

On average, XO units are used for 17 years – equivalent to more than 30,000 working hours.

Therefore, acquiring a new unit is one of your most important investment decisions.

XO FLOW helps you work smarter, on calm patients, while staying healthy. Probably the best investment you'll ever make.

The cost of ownership – be it depreciation, interest or maintenance – pales in comparison to the extraordinary benefits.

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STANDARD CONFIGURATION

XO FLOW is a unit for both newcomers and experienced users of XO units.

It is easy to monitor and control all unit and instrument functions due to the Dashboard – a touch-activated screen integrated with the unit's instrument bridge.

As an existing XO user, you can operate the unit handsfree – as you are used to – using the foot control.

Configure XO FLOW with exactly the instruments, features, colors and finishes that you require.

See all the possibilities from page 22.

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Se alle mulighederne fra side 22.

ADVANCED CONFIGURATION

The advanced XO FLOW configuration includes software options that enable multiple users to work with individual instrument presets and to access XO dental apps on the Navigator – a second touch-activated screen.

Presets make it easy – with just one tap – to select optimal instrument settings for a specific dental procedure.

Unit apps are used for setting up and managing the unit, for handling images/videos and sending e-mails etc.

Remote desktop apps make it possible to interact with software running on one or more remote computers.

Web apps connects directly to web-based solutions.

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EASY TO MONITOR AND ADJUST INSTRUMENT DATA

When you lift an instrument, the control window opens.

With the little finger of the hand holding the instrument you can for example use the “red” slider to adjust the maximum speed.

The automatic chip blow provides a constant clear eyesight each time the instrument stops – without changing to/from the syringe. Enable the automatic chip blow with .

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SCREEN OPTIONS

Choose between the Navigator with a long and flexible arm – or a 27" UHD screen on a short arm.

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THE NAVIGATOR

The Navigator is a touch-activated screen on a flexible arm making it possible to position the screen near the treatment area.

The Navigator can be used to monitor and control software running on a praxis computer when this is connected to the same local area network as the unit.

An advanced configuration with XO dental apps is realized when an XO FLOW unit with Navigator is enhanced with software options. See more from page 74.

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UHD screen

Choose between eleven patient chair colors, standard or soft cushions, ten Comfort fabric colors and five skai colors.

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Configure your XO FLOW unit
with – or without cuspidor and
cup filler.

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SUPERB INFECTION CONTROL

It is simple and fast to clean and disinfect the unit.

At startup/shutdown the water and suction lines are automatically disinfected.

It is easy to remove all handles, suction nozzles, cuspidor bowl, cup holder and the instrument holder for disinfection.

All surfaces are smooth, and the metal parts are coated with antibacterial powder paint.

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The instrument holder is fitted with magnets and the cover can easily be exchanged and autoclaved.

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Place the instrument bridge in the working position over the patient's chest. The patient neither sees nor touches the instruments.

The chair is extremely comfortable with ideal support of the neck and arms.

The cushions may be fitted with XO Comfort fabric that allows the patient's skin to "breathe".

The chair has an automatic safety stop.

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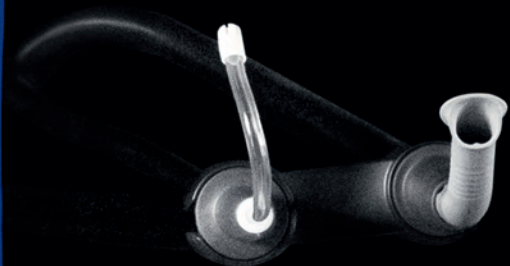
AMBIDEX

After use, the suction hoses are automatically repositioned in the suction hose holder.

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SAVE TIME: WORK DUO

Work fourhanded and increase the productivity.

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Vær din egen assistent.

SAVE COSTS

Fit the unit in a treatment room of only 6 m². Save rental costs or design the clinic with more treatment rooms and increase patient turnover.

The unit only needs preventive service every second year.

The built-in TeamViewer application makes it possible to connect online to a remote service technician for monitoring and intervention.

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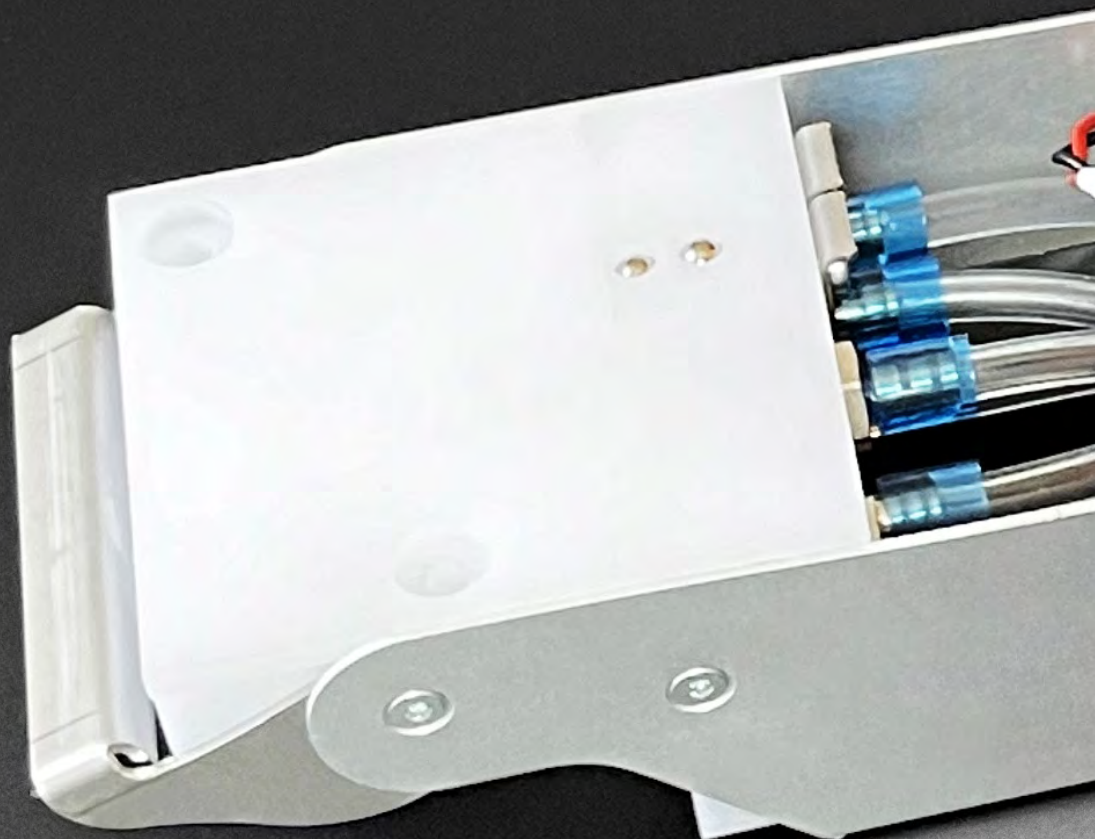
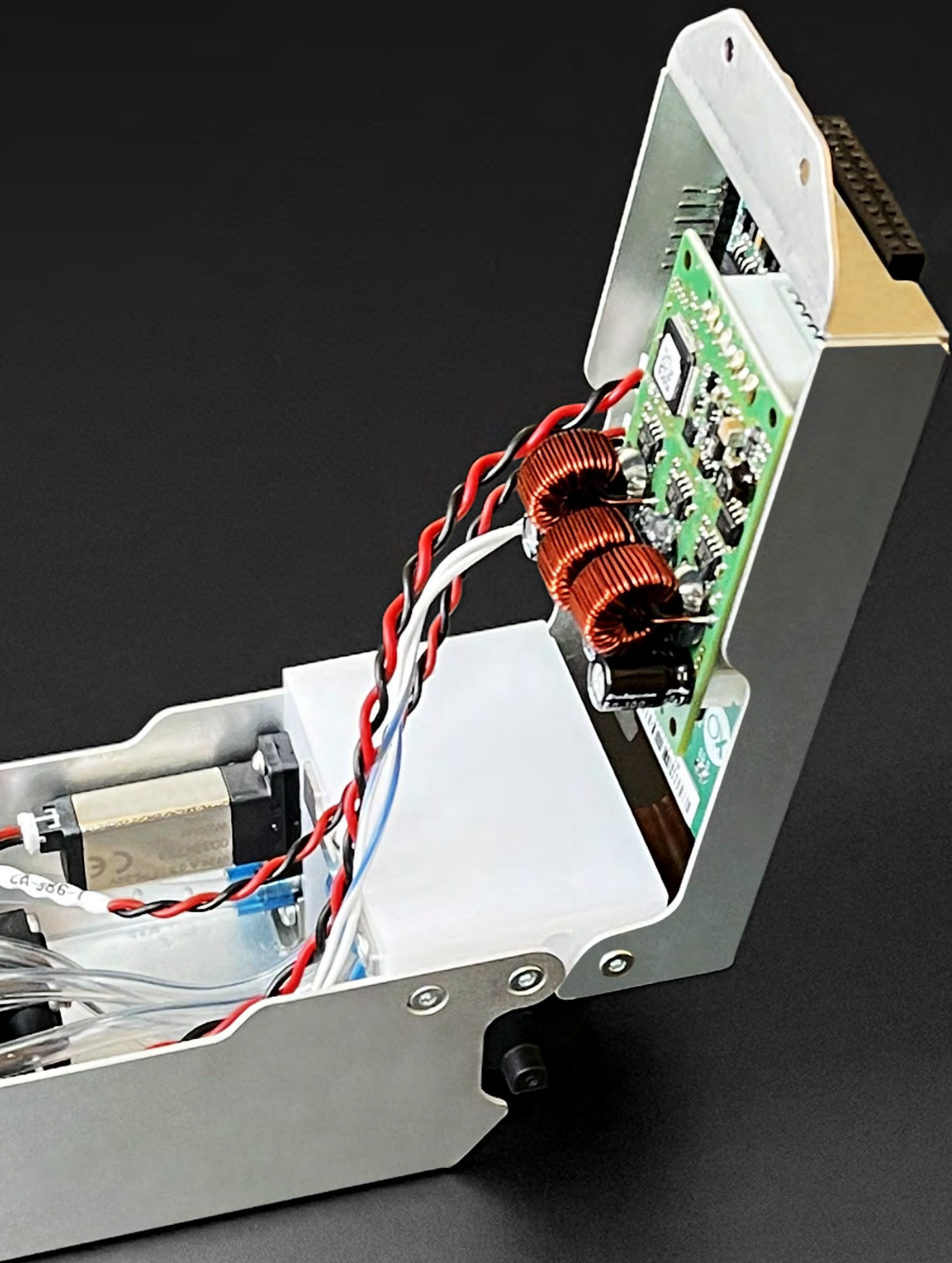
Din garanti for høj opetid.

The dental industry is undergoing significant change as practices become larger with many – often part-time – employees.

Practitioners are requesting connectivity solutions with integration of the units with software applications.

Finally, health authorities are demanding more standardization and documentation of, for example, infection control procedures.

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ADVANCED MULTI-USER DENTISTRY

To cater for the new trends, XO FLOW is available in an advanced multi-user version including the Navigator and software options featuring an intelligent user interface with XO dental apps and instrument presets.

The apps are used to set up and control the unit and to integrate with clinic software applications.

Presets make it easy to select ideal settings for a specific procedure with a single tap.

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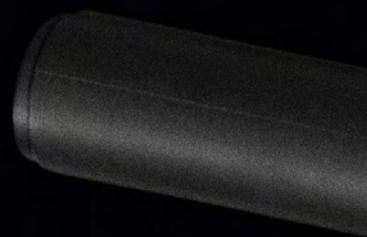
▶▶▶

XO DENTAL APPS

In a fully configured XO FLOW these XO dental apps are available:

- **Remote desktop apps** (☒) for managing software running on remote computers – e.g. intraoral scanners and practice management systems
- **Web apps** (☁) for accessing and managing web-based software solutions – e.g. Netflix and practice management systems
- **Unit apps** for setting up and managing the unit, handling images/videos and sending e-mails etc.

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
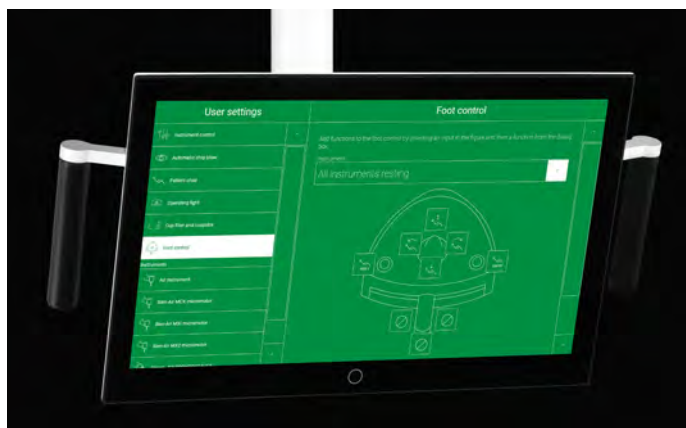
Open an app from the Navigator home screen with one tap.

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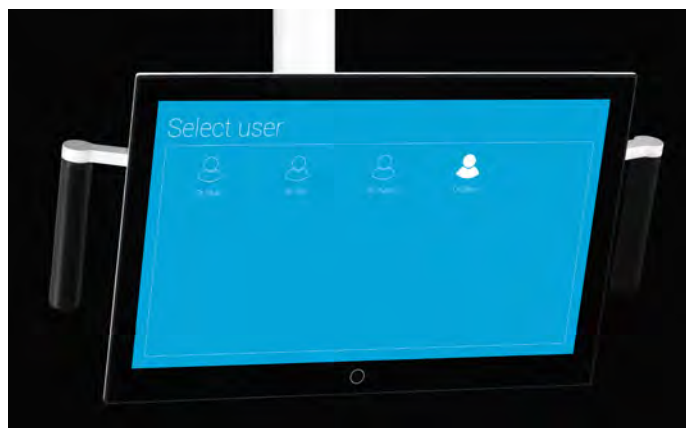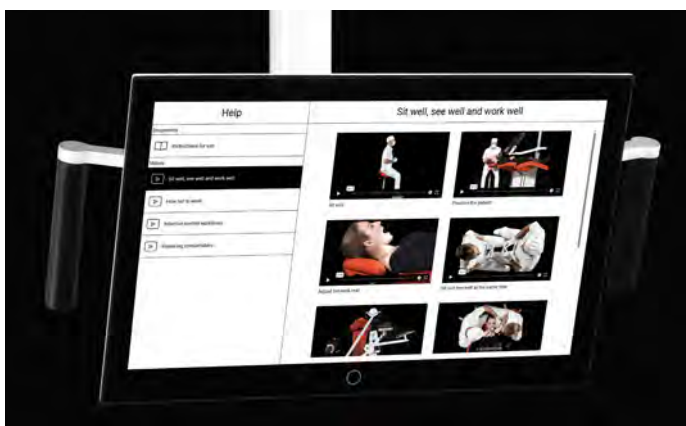
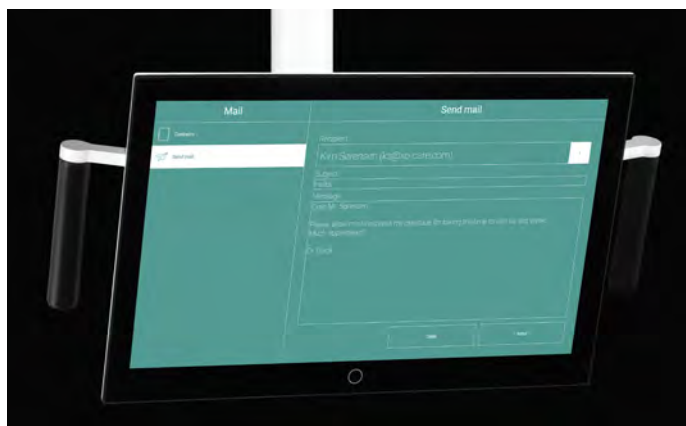
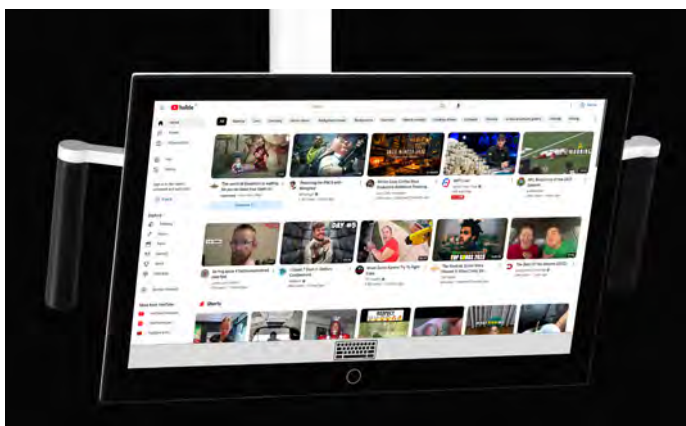
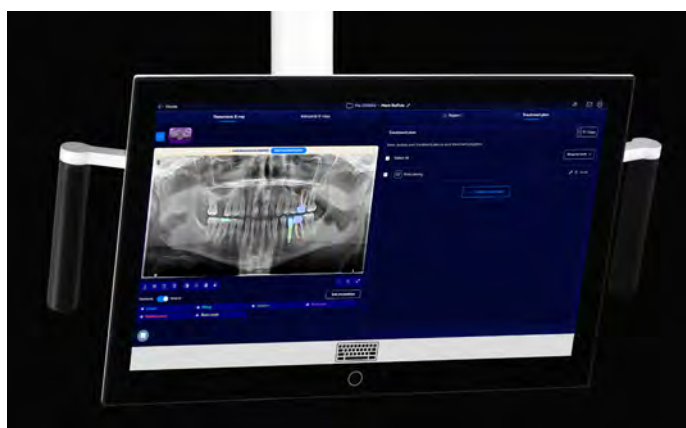
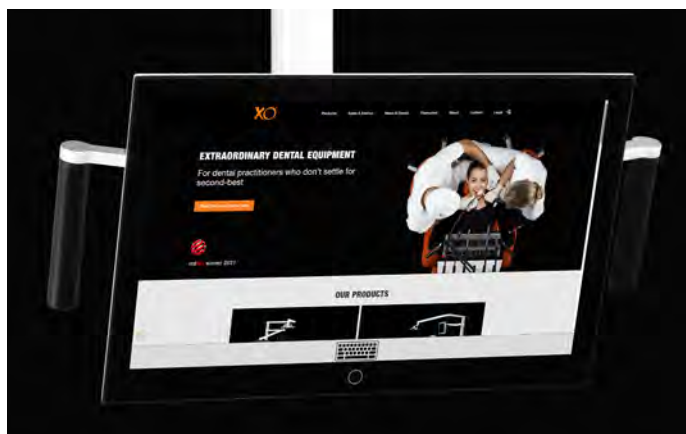
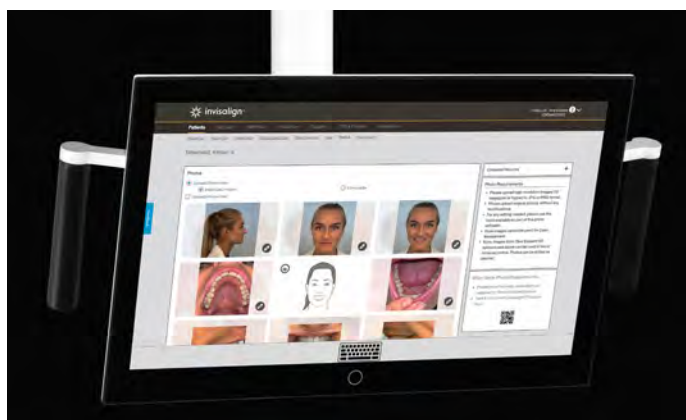
Invisalign® ☁

iTero Element™ Flex 🖥

INTEGRATION WITH INTRAORAL SCANNERS

With XO FLOW it is easy to integrate with and manage intraoral scanners directly from the Navigator.

You avoid investing in stand-alone digital devices and you get:

- Increased focus on the patient
- Healthy working postures
- Easier infection control
- More space in the treatment room

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XO CARE

MORE THAN 70 YEARS OF EXTRAORDINARY DENTISTRY

Ever since Verner Sørensen started the company in 1951, XO CARE has been committed to support dentists improve their professional life.

To this day, we remain pioneers in the design of EXTRAORDINARY dental equipment and instruments.

We develop and manufacture the XO products in a refurbished and top-modernized historic setting in Hørsholm, Denmark.

Both the second and third generation of the Sørensen family are still active in the company:

Kim Sørensen as CEO and Chief Designer

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Kim Sørensen come CEO e Chief Designer

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Peter Sørensen som CTO

DESIGN THINKING

At XO CARE we take pride in creating meaningful and extraordinary dentistry solutions. We do this by combining our expertise and innovative ideas with the latest technology.

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The ID Prize
1982

The ID Prize
1986

The European
Community
Design Prize
1988

The Danish
Design Prize
2007

Red Dot Winner
2021
in category
"Medical
Devices
and
Technology"

Red Dot Winner
2021
in category
"Innovative
products"

C€ 2460

All rights reserved.

XO® and XO FLOW® are registered
trademarks.

© XO CARE A/S 2024.

XO FLOW is a combined unit and
patient chair which is to be used
by dentists and other qualified
personnel for prevention and
treatment of human teeth diseases.

Issued and pending patents.

See technical and legal product
details at xo-care.com.

Please note that not all the described
features are included in the standard
version of the XO FLOW unit.

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